

FDA clearance for MagVenture: Cut your treatment time in half...at no extra cost!

The standard, 10 Hz / 3,000 pulse, FDA cleared protocol* can now be done in half the time!

MagVenture has received 510(k) FDA clearance for:
"... a range of inter-train intervals from 11 to 26 seconds, rather than the fixed 26 second duration, which will allow a reduction in treatment time from 37.5 minutes to a minimum of 18.8 minutes."

The total number of pulses as well as trains, the intensity, repetition rate and everything else is the same – all you have to do is reduce the aforementioned Inter Train Interval (ITI) between the 75 pulse trains, and you can go down to as little as 18.8 minutes per session. Yes, it really is that simple!

Your MagVita TMS Therapy System** can – thanks to the extremely efficient liquid cooling system and coil – easily handle the reduced ITI with no risk of overheating. In other words, you can start reducing your treatment time TODAY - at NO EXTRA COST! The box on your right explains how.

Please do not hesitate to contact us at 888-624-7764 / infousa@magventure.com if you have any questions.

**The MagVita TMS Therapy[®] system is indicated for the treatment of Major Depressive Disorder in adult patients who have failed to receive satisfactory improvement from prior antidepressant medication in the current episode.*

***with a MagPro R30/X100 stimulator.*

How to add the 19 minute TMS treatment session to your MagPro device:

1. Recall/load your FDA cleared protocol.
2. In the main menu press "timing".
3. Select Inter Train Interval (ITI).
4. Set the ITI to 11 seconds.
5. Press configure.
6. You may now save the new 19 minute protocol either as "default" or under a letter from A-Z.

Please note

Please read the complete user guide including warnings, precautions, and side effects prior to use.

